



Assure®

V13 2/18

UNIVERSAL BONDING RESIN FOR WET OR DRY ENVIRONMENTS

INDICATIONS FOR USE: Assure® Universal Bonding Resin is intended to increase adhesion to wet or dry, normal and atypical enamel surfaces including: hypocalcified, fluorosed, deciduous enamel and dentin. It will bond to roughened metal (amalgam, gold or stainless steel) and composite restorations without additional primers.

PRODUCT FEATURES: Assure® Universal Bonding Resin is hydrophilic so the etched enamel can be dry or slightly contaminated with saliva. Assure® Universal Bonding Resin does not require light activation (except on dentin). It can be used under any chemical, light cure or dual cure paste, regardless of manufacturer.

DIRECTIONS FOR USE:

PROPHYLAXIS: Using a rotary instrument with a rubber cup or brush, prophylax the enamel surfaces to be bonded with 1st & Final® or plain, oil-free pumice. Rinse and dry thoroughly.

ETCH: With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Allow 30 seconds for etching (60 for deciduous teeth). Avoid placing etching agent on soft tissue. Note: If using a self etching primer in place of phosphoric acid - scrub the SEP into the enamel for 5 seconds, air dry thoroughly, apply Assure, dry with air and proceed with bracket placement.

RINSE & DRY: Rinse and dry thoroughly. The etched area should appear frosty white. If not, re-etch for an additional 20 seconds.

APPLICATION OF BONDING RESIN: With a brush or sponge pellet, apply one coat of Assure® Universal Bonding Resin to each prepared tooth and stroke over to ensure penetration into the enamel and evaporation of the solvent. No light curing of the sealant is required, with the exception of dentin. If after drying the enamel does not appear shiny, apply an additional coat of Assure® and dry. Proceed with the application of any chemical or light cure paste with bracket.

BONDING TO A COMPOSITE RESTORATION: Condition any exposed enamel as previously described. Sandblast or roughen the composite surface with a fine diamond. Rinse and dry the area, apply one coat of Assure® Universal Bonding Resin and lightly dry. Proceed with the application of paste and bracket.

BONDING TO AN AMALGAM RESTORATION OR METAL SURFACE: Condition any exposed enamel as previously described. Sandblast and apply one coat of Assure® Universal Bonding Resin and lightly dry. Proceed with application of paste and bracket.

BONDING TO PORCELAIN: Prophylax, rinse and dry. Sandblast, rinse and dry. Apply one coat of Porcelain Conditioner, wait 60 seconds. Lightly air dry. Apply one coat of Assure, air dry and light cure for 10 seconds. Proceed with bracket placement.

Special Safety Notice: Brush applicators are single use devices. It is required that a new brush applicator is used for each application.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



Flammable.

Reorder Information: REF# AS (6mL), REF# AS2ML (2mL)



TEMPERATURE LIMITATION: Store at room temperature (15-30 Degrees C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: Federal law restricts this device to sale or on the order of a dental professional.



Reliance Orthodontic Products, Inc.



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com



Scan for more information

Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands