



V12 4/18

RELIANCE PLASTIC APPLIANCE CONDITIONER

REF PL

INDICATIONS FOR USE: The **Reliance Plastic Appliance Conditioner** is intended for use as a conditioner of plastic brackets or acrylic appliances in preparation for bonding with any adhesive, sealant or cement. It can also be used to condition a composite restoration or acrylic jacket prior to bonding and to rebond a bracket impregnated with adhesive.

INSTRUCTIONS FOR USE:

BONDING A PLASTIC BRACKET OR ACRYLIC APPLIANCE:

With a brush or sponge pellet, apply a thin uniform coat of **Reliance Plastic Appliance Conditioner** to the plastic surface to be bonded. Wait at least 60 seconds before applying the bonding paste.

BONDING TO A COMPOSITE RESTORATION:

With a fine diamond, lightly abrade the composite to remove the shine. Clean area with an air syringe. Using a sponge pellet or brush, apply a thin uniform coat of **Reliance Plastic Appliance Conditioner** and allow to dry. Apply a thin coat of bonding resin to the conditioned composite surface and the paste to the bracket. Place bracket on tooth and position. If Rely•a•Bond® is the adhesive of choice, apply the Rely•a•Bond® Primer to the conditioned composite and the bracket base. Apply the paste to the bracket base, place the bracket on the tooth and position.

NOTE: A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical

attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



TEMPERATURE LIMITATION: Store at room temperature (15-30 Degrees C).

Avoid excessive heat.



Flammable



Harmful to Aquatic Life.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Svdnev. NSW 2000 Australia

Scan for more information

Check out our new learning platform Reliance
U!
www.RelianceU.tech